

ADM's OxyHealth Connection Kit

1: To connect to the chamber

- Remove the brass bolt, at the head of the OxyHealth chamber (210, 270, 320).
- Insert the short end of the tubing through the opening where the bolt was.
- Screw in the brass fitting (already on the tubing) into the chamber snugly (finger tight).



2: To connect to the concentrator

- The 'Christmas tree' connector screws onto the output on the concentrator.
- Connect the long end of the tubing to the 'Christmas tree' (push on)



Using the Concentrator with HBOT

Always turn on the concentrator 1st for 3-5 minutes to allow it to 'warm up' and get the pressure and oxygen up to therapeutic levels. Alarms, buzzers and lights will go off after a few minutes. Then get in the chamber.

Note:


Always keep the tubing untangled and un-kinked to ensure air flow.

Keep the tubing free from walkways as it may cause a trip hazard.

Connecting a Mask or Cannula to tubing inside the chamber

No adapter needed:

Insert the cut end of the tubing directly into the female end of the mask or cannula.

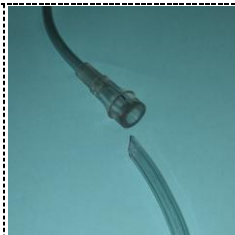

Extra parts:

The short 'end' piece can be used to connect additional items needing a female/female connection.

The male/male connector can be used to connect 2 female connections.

Sometimes these 2 parts come connected.

Contact us if you have any questions: 800-877-9100 or Sales@AmericanDiscountMed.com