

ADM's IMPORTANT NOTES

Drive Devilbiss 1025DS

- **REMOVE THE PLASTIC WRAP BEFORE USE**
- When you get the unit unpacked, let it acclimate to the inside temperature before using, especially in cold weather.
- On each first use, plug it in without any tubing attached and let it run for 10-15 minutes. The alarm will sound for a few seconds as it self-tests. Allow the unit to 'rev up' and get the O2 to purity. Oxygen purity light will change from yellow to green in about 90 seconds.
- When connecting the concentrator to a chamber, always turn on the concentrator first for a 2-3 minute and let it warm up first, then pressurize the chamber. When therapy is completed, decompress the chamber, then turn off the concentrator last
- Ideally when using the concentrator with a chamber, always keep the LPM flow rate at 8/9 to get the optimal oxygen percentage. Never above 10 LPM.
- The Drive concentrator uses 6.1 Amps of electricity. Be sure to plug the unit into a dedicated outlet to insure maximum effectiveness in use. **No extension cords!**
- The air intake/output is on the back and underside of the concentrator. For this reason place the unit on a hard floor surface (or very short carpet) to ensure good air flow.

There is 1 user changeable air filter on this unit.

- The internal 'box' filter is in the rear behind the right side of the unit, behind the filter door. It should be replaced annually depending on the environment.
- A final bacteria filter is inside the unit and cannot be changed w/o disassembly.
- There is no external mesh filter on the rear of the unit. Re-design removed this filter.

Please contact us if you have any questions or concerns.
American Discount Medical - 800-877-9100, sales@AmericanDiscountMed.com