

ADM's IMPORTANT NOTES

NIDEK 595

- **Install the caster before plugging the unit in.**

- When you get the unit unpacked, let it acclimate to the inside temperature before using. Especially in cold weather.
- Before 1st use, double check the internal box filter is in place located behind the rear foam filter (right side, may have been dislodged in shipping)
- On initial use, plug it in without any tubing attached and let it run for few minutes. This will allow the unit to 'rev up' and get the O2 to purity. 'OK' Blinks when getting ready, 'OK' Solid when ready
- When using the concentrator with a chamber, always turn on the concentrator first for a 1 minute to create the oxygen and build up pressure... then turn on the chamber.
- When using the concentrator with a chamber, always keep the **LPM flow rate** per your doctor's instructions, **usually around 8-10 LPM.**
- The concentrator uses approximately 5 amps of electricity. Be sure to plug the unit into a dedicated outlet to insure maximum effectiveness in use. **No extension cords or power strips!**
- Do not cover the rear air filter on the back of the unit. The unit pulls in fresh air from this filter. (ex: do not place items on concentrator like a jacket or blanket that might obstruct the air flow)
- New units may have a burn-off smell. This will go away after a few hours of running the unit. Not problem, this is normal.

There are three (3) filters on this unit.

- The rear foam filter can be found on the back of the unit; it should be removed and cleaned with mild soap and water weekly, let dry over night before reinserting.
- The internal 'box' filter is on the back of the unit behind the right foam filter. It should be replaced annually depending on the environment.
- The bacteria filter is inside the unit (looks like a top). The case must be opened to access this filter. This filter does not usually need replacing unless servicing or replacing the compressor

Filter change information is from the Nidek Service Manual p.56 can be found on our website.

Please contact us if you have any questions or concerns.
American Discount Medical - 800-877-9100, sales@AmericanDiscountMed.com